

PATIENT IDENTIFICATION WRISTBANDS



2013 CATALOG

Patient Identification Bands

Patient ID bands is an important way to reduce patient safety risks. Patient misidentification continues to result in medication errors, transfusion errors, testing errors, wrong person procedures, and the discharge of infants to the wrong families. Today, with hundreds and thousands of patients being admitted and discharged from hospitals and nursing homes every day; identification of patient using wristbands is an integral component of smooth running and helps minimize medical errors and administrative issues can be handled efficiently.

Bio-X offers optimal and secure patient identification solutions for healthcare facilities. Colour coding of wristbands facilitates rapid visual recognition of specific issues and can be modified as per the requirement of the organization.

We offer Patient Identification solutions for admissions, blood, mother and baby, paediatrics, disaster response, outpatient and emergency room purposes.

USING PATIENT IDENTIFICATION BANDS


The following information may assist practitioners in the correct identification of patients upon admission to their institution. This is a series of recommendations only and is by no means comprehensive. Practitioners should consult the policies and procedures of their institution regarding the process and practice of identifying patients.

1. **Identify the patient** according to approved practices.
2. **Select the correct identification band size and colour to be used** according to the requirements of the patient and the policy of the institution.
3. **Apply the identification information to the band** while in front of the patient
 - Ensure the information is clear and legible, especially if hand-written
 - Ensure the main identifiers are clearly visible: Name, Date of Birth and Medical Record Number.
4. **Apply the band to the dominant wrist** as intravenous cannulation is often first performed on the non-dominant side
 - Use the other wrist if intravenous therapy is already in progress
 - Ensure one finger space underneath the band to prevent the band being too tight or too loose
 - Apply a second band around the ankle on the same side if this is the practice in your institution
5. **Remove excess strap length** for improved patient comfort.
6. **Observe each shift for the following:**
 - Missing or damaged band, especially at the closure point
 - Damaged or faded information or information field
 - Constriction by the band due to limb swelling, etc
 - Loose fitting of the band that may facilitate removal.
7. **Replace the identification band immediately:**
 - If the existing band is removed for any reason such as intravenous therapy or other procedure
 - If blood or other body fluids spill onto the band.
8. **Change the band at least one each week** to maintain the legibility of the information on the band and to prevent complications such as infections.


Mother Infant Bands

Insert card Type

- Insert Card
- Soft Vinyl
- Snap Closure
- Custom Printing Available on Request
- Information Area: 65mm X 21mm (Adult)
53mm X 15mm (Infant)
- Packing : 250 sets / box

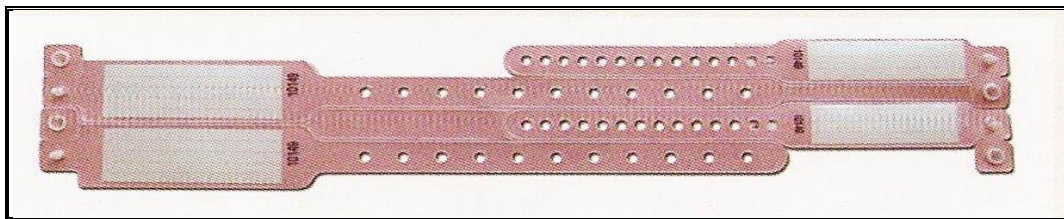

Colours:

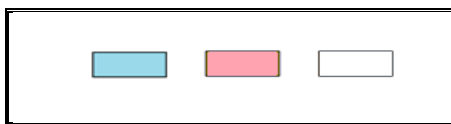


Blue Pink White


Write on Type Bands

Write on Type Bands

- Soft Vinyl
- Snap Closure
- Custom printing on Request
- Information Area : 54mm X 18mm (Pediatric)
- Packing : 250 pcs / box

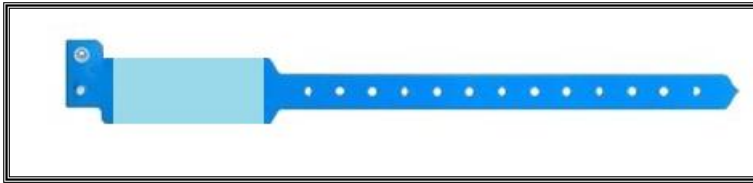

Colours:

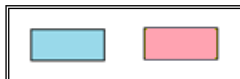


Blue

Pink

Insert Card Type Bands

Insert Card Type Bands

- Soft Vinyl
- Snap Closure
- Can fit upto 6 lines of information
- Custom printing on Request
- Information Area : 87 mm X 20mm (Adult)
48mm X 15mm (Pediatric)
- Packing : 250 pcs / box

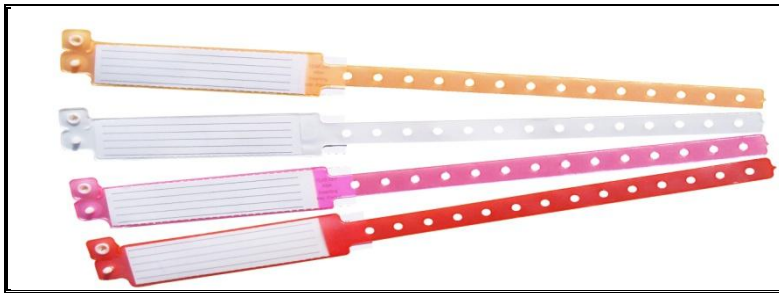

Colours:



Red Green Blue Black Pink White

Contact Person : Mr. Pravin Gawande

Amey Incorporation

Pune 411041

Cell : 09423439973

Email :pravin@ameyinc.com
